

cPL

Canine pancreas-specific lipase



SPECIAL FEATURES

- Direct measurement of cPL concentration
- Now, start to measure dog's pancreatic lipase at your hospital!

REFERENCE RANGE (Measurement Range 50~2000ng/ml)

NORMAL ≤ 200 ng/ml

EQUIVOCAL 200~400 ng/ml


ABNORMAL 400~2000 ng/ml



SAMPLE VOLUME

3 STEP PROCEDURE

- 1 Get 25 μ l of serum by using the capillary spuit

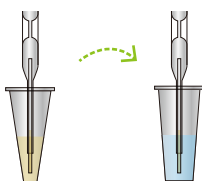

- 2 Using the tablet capillary spuit, mix well



- 3 Drop the mixed sample onto the kit

